



Online store WWW.LYNXGEAR.LV
Company: SIA LATLYNX
info@lynxgear.lv, +371 26159668
Reg. No. 45403038162, VAT Reg. No.
LV45403038162,
Legal address: Baznicas iela 8,
Aizkraukle, Aizkraukles novads,
LV-5101, LATVIJA
Bank Account: LV87UNLA0050021387494
AS SEB Bank, SWIFT: UNLALV2X

ORDER CONFIRMATION

Not Rated Yet



Sales price 27,95 €

Salesprice with discount

Incl. VAT 21%: 4,85 €



Description

Rhino Rescue Wound Closure Kit

Instant, Painless Laceration Management • Compact First-Aid Solution

Close cuts and lacerations **seamlessly & pain-free** - wherever you are. The Rhino Rescue Wound Closure Kit puts a mini "ER" in your pack, supplying everything required to disinfect, approximate and dress small wounds without needles, sutures or skin glue. Ideal for *EDC medical pouches, vehicles, range bags, travel or home preparedness.*

Kit Contents (21 × 24 × 2.5 cm · 165 g)

- **6 × Adjustable Wound-Closure Strips** (*two tension tabs let you draw wound edges together; use multiple strips for longer cuts*)
- 1 × BZK Antiseptic Pad
- 2 × Alcohol Swabs (70 % IPA)
- 1 × Sterile Gauze Pad & 1 × Roll Bandage
- 1 × Transparent Waterproof Dressing
- 1 × Pair Nitrile Gloves • 1 × Trauma Shears • 1 × Roll Adhesive Tape
- Illustrated **Instruction Manual**

Why Choose Rhino Rescue Closure Strips?



Online store WWW.LYNXGEAR.LV
Company: SIA LATLYNX
info@lynxgear.lv, +371 26159668
Reg. No. 45403038162, VAT Reg. No.
LV45403038162,
Legal address: Baznīcas iela 8,
Aizkraukle, Aizkraukles novads,
LV-5101, LATVIJA
Bank Account: LV87UNLA0050021387494
AS SEB Bank, SWIFT: UNLALV2X

ORDER CONFIRMATION

- **Stitch-Free** – bridge wound edges without needles or anaesthetic.
- **Adjustable Tension** – twin pull-tabs let you fine-tune closure pressure.
- **Secure Adhesion & Durable Fleece Backing** keep the closure in place during healing.
- **Rapid Deployment** in field, travel, sport, camping, range, workshop or disaster kits.

How to Use (summary)

1. Put on gloves. Clean wound area with alcohol swab & BZK pad.
2. Apply sterile gauze to blot excess blood.
3. Peel backing and apply closure strip so each side adheres firmly on dry skin.
4. Pull tension tabs until wound edges meet; press tabs down to lock.
5. Cover with transparent dressing or gauze & bandage if additional protection needed.

Note : For deep or heavily contaminated wounds, severe bleeding or signs of infection, seek professional medical help immediately.

Package Specs

Dimensions (packed)	≈ 21 × 24 × 2.5 cm
Weight	≈ 165 g
Shelf-Life	See date stamp on outer pack
Intended Use	Field first-aid for minor lacerations / cuts

Safety & Compliance

Use only on clean, superficial wounds no wider than the closure strips. Check skin for adhesive allergies. Single-use, sterile items – do not reuse. Follow enclosed IFU (Instructions For Use). Store in a cool, dry place.

Manufacturer | GPSR Responsible Person

EMS RUN / Rhino Rescue

No. 100 Fenbin Rd · 214000 Wuxi City · China

emsrun.com/contact